

B-1. Prescription Drugs Giving Positive Results for the T-Cup test

The list of brand names is not complete

AMPHETAMINE, METHAMPHETAMINE

Generic Name	Brand Names	Notes
Amphetamine	Adzenys, Dyanavel XR, Evekeo	
Amphetamine/ dextroamphetamine	Adderall, Adderall XR, Mydayis	
Benzphetamine	Didrex	metabolizes to amphetamine and methamphetamine
Dextroamphetamine	Dexedrine	
Lisdexamfetamine	Vyvanse	Metabolizes to dextroamphetamine
Methamphetamine	Desoxyn	

BARBITURATES

Amobarbital	Amytal	
Butobarbital	Butisol Sodium	
Butalbital	Found in Fiorinal, Fioricet, and many others	Sedative. May be found in combination prescription drugs used to treat migraines or other headaches.
Pentobarbital	Nembutal Sodium	
Phenobarbital	Luminal	
Secobarbital	Seconal Sodium	

BENZODIAZEPINES

Generic Name	Brand Names	Notes
Alprazolam	Xanax	
Clobazam	Onfi	
Chlordiazepoxide	Librium	
Clonazepam	Klonopin	
Clorazepate	Tranxene	
Diazepam	Valium	
Estazolam	Prosom	
Flunitrazepam	Rohypnol	
Lorazepam	Ativan	
Midazolam	Versed	
Oxazepam	Serax	
Temazepam	Restoril	
Triazolam	Halcion	

OPIATES

Generic Name	Brand Names	Notes
Codeine	Tylenol #3, Guiatuss A.C., Robitussin-AC, Triacin-C	Products with codeine in the brand name are not listed.
Hydrocodone	Norco, Tussionex, Vantrela ER, Zohydro ER	
Hydromorphone	Dilaudid, Exalgo	
Levorphanol	Levo-Dromoran	
Morphine	Arymo ER, Astramorph, Duramorph, Kadian, Morphabond ER, MS Contin,	
Oxycodone	Oxaydo, Oxycet, OxyContin, Oxydose, OxyFast, OxyIR, Percodan, Percocet, Roxicet, Roxicodone, Roxybond, Xtampza ER	
Oxymorphone	Opana	

Notes: Meperidine (Demerol) does not cause a positive opiate reaction with this test.

THC

Generic Name	Brand Names	Notes
	Marinol	

Note: This table reflects information reported by test kit manufacturer. Clinical judgement must be used in considering the half life of the drug and the timing of the last dose prior to the urine test.